


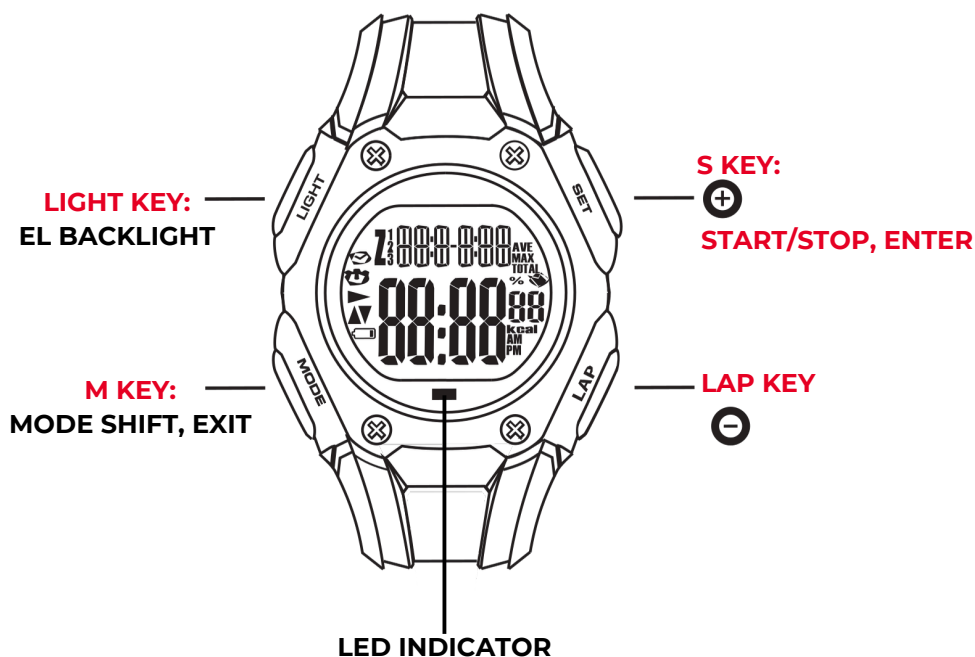

## ENGLISH

### ► HEART RATE MONITOR AND ACCESSORIES



Heart Rate Transmitter



CR2032 3V Lithium battery of Transmitter




Battery Cover of Transmitter



Battery Rubber Pad of Transmitter (O-ring)

### ► WATCH

Use M Key to switch operating modes:

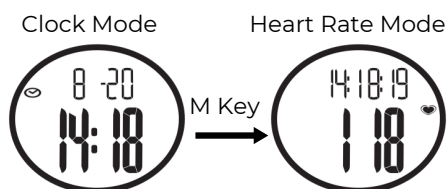


The watch has two main modes and screens: clock mode (with time and date) and heart rate mode.

To switch from one mode to the other, simply press the M button.

**IMPORTANT: When you do not need to detect the horse's heart rate always return to the main screen – CLOCK MODE with time and date.**

## ► HOW TO SET DATE AND TIME

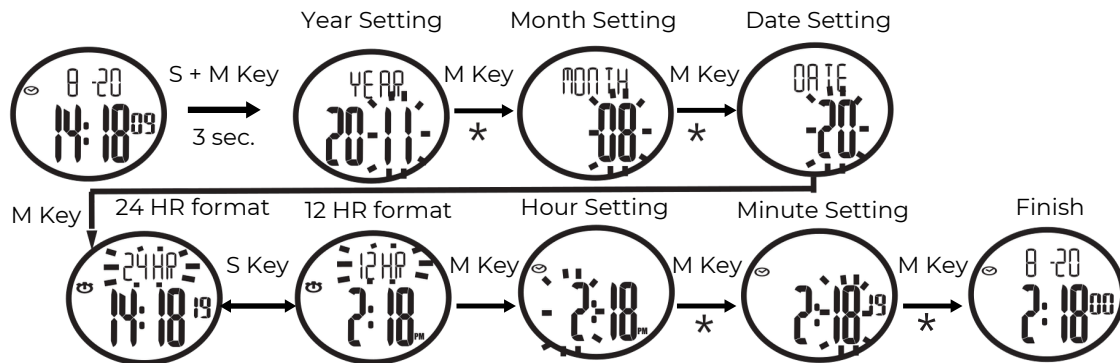

Press the S KEY first and press the M key simultaneously for 3 seconds, the digits “YEAR” will flash. To advance the year, press the S KEY until you get the year that you want. Press the LAP KEY to decrease for setting the year.

Press the M KEY to next setting and adjust the MONTH then press M KEY and adjust the DATE with the same process above. Press M KEY and choose between “12HR” or “24 HR” by press the S KEY or LAP KEY to shift clock display to 24 hour or 12 hour format. Press the M KEY to next setting to adjust HOUR and MINUTE, follow the same process above. Press the M KEY to finish clock setting and enter clock mode.

## ► HEART RATE DETECTION

Press the M key to shift from clock mode to heart rate mode.

### Heart Rate Mode:

Simple HR display



Upper line is the time of day.

The ❤️ symbol on the watch will flash and display the current heart rate value.

Before each use and race, check that the handle is working properly. Place two fingers on the sensors and check that a Red Light flashes on the handle.

**Video >** <https://www.youtube.com/watch?v=N-dJzzRy1r8>

1. Do not immerse the handle in water, but wet the horse thoroughly.
2. Place the handle correctly on the horse, making sure the sensors are well in contact with the skin.

Click on the link to see the correct position of the handle >

<https://www.instagram.com/reel/C5yhmJSI6b9/>

3. Press the mode button on the watch to switch from time mode to heart rate detection mode
4. Heart detection, if the handle is positioned correctly and the horse is properly wet, occurs in 4-6 seconds.
5. When the frequency detection is finished, press the MODE button and return to the main screen.

The HR monitor may not detect frequencies below 30 beats.


## ► IRREGULAR BEATS

In case of irregular beats, check that the handle position is correct, and the horse is well wet.

If the problem persists, perform a RESET. Press the M and S button simultaneously for 8 seconds.

## ► HOW TO REPLACE THE HANDLE BATTERY

USE METAL FORCEPS

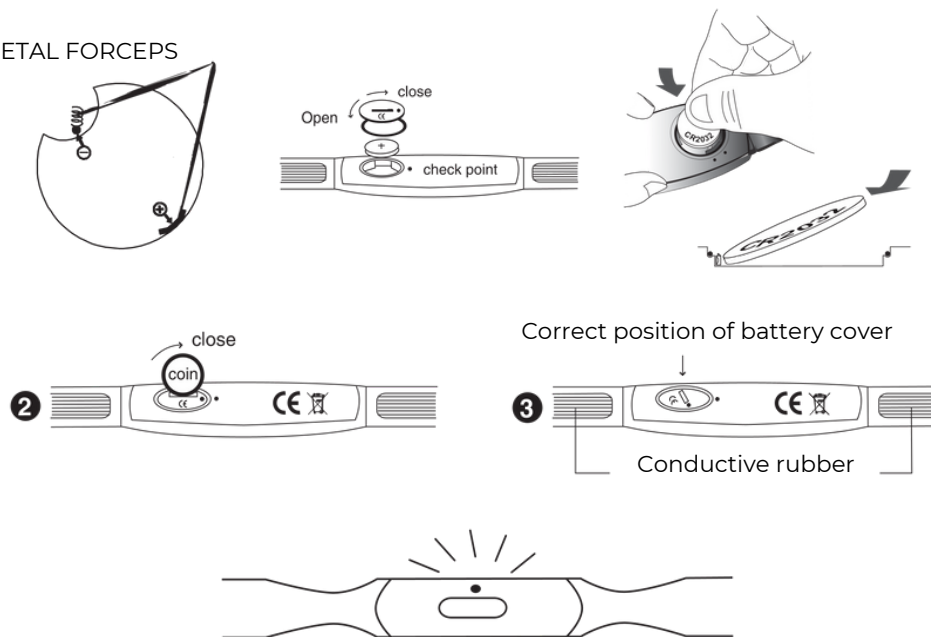

Remove the non-functioning battery and insert a new CR2032 battery with the "+" sign symbol facing upward. Make sure the red gasket stays inside the battery space, insert the new battery, and put the black cover on by making 3/4 turn with a coin. Reapply the clear film.

**EQUITIME**

### Distributed by:

Equitime SRL  
Viale Dei Pini 10  
Cepagatti (PE) 65012  
ITALY  
info@equitime.it  
www.equitime.it